ADHESIVES ADESIVI

MINI MOLD

64

CEMENTO PER BANDE ~ BAND CEMENT

BLUE-BAND

66

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Cod FC-MMKIT

Mini Mold is an useful and easy-to-use product, ideal to create composite attachments of different shape, directly on the tooth. Mini Mold is made of several capsules of different shapes, one holder for the direct application and a step-by-step Instructions sheet. The use of our FC-BOND-008 adhesive is recommended.

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10 capsules + 1 handle + detailed instructions

MINI MOLD KIT

27 capsules, 3 of each style + 1 handle + detailed instructions

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HABIT-CONTROL

To create relief that ends thumb sucking or tongue thrusting.

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BITE-RAMP

To create a bite opening ramp on the lingual of the up anterior teeth. Available in small (3mm) or large size (5mm).

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POSTERIOR MOLAR PAD

Designed to open the bite from the posterior teeth.

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BUTTON

To create easily and quickly low profile buttons.

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WIRE BONDER

Arch and wire anchorage to make lingual retainers. Available in small or large size.

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BRACKET

To make small edgewise brackets for simple teeth movements.

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To create temporary tubes.

cod FC-MM06

SEMPLICE TECNICA IN 2 FASI ~ SIMPLE TECHNIQUE IN TWO STEPS

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STEP ONE ~ Choose the capsule between the 9 availables and fill it with the light cure composite.

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STEP TWO ~ Position the capsule on the tooth and polymerize for 15-20 seconds. Take off the capsule and go on with the polymerization for 20 seconds more.

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The Ultra Band Cement offers optimum elastic properties without the need of being premixed. It allows an exact dose that can be applied directly on the band. The easy activation of the material reduces the working time. Excess flash can be easily cleaned-up. The composition of the product provides a better strength compared to conventional powder/liquid glass ionomer and zinc phosphate. Its fluoride releasing reduce the chance of decalcification. The blue color make easy to remove the cement excess.

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KIT

- › 4 x 5 g syringes
- › 2x3 ml of resin
- › 7ml of etchant bottle
- › tray and brushes

REFILLS

- | | | |
|-----------|--------|-------------|
| › Syringe | 5g | FC-BOND-017 |
| › Resin | 2x3 ml | FC-BOND-019 |
| › Etchant | 7 ml | FC-BOND-030 |

Polymerization: 20 seconds working time using a led lamp. 20 second working time with a led lamp.

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