ACCESSORI ACCESSORIES

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Our bondable accessories are made of medical-grade steel. The bondable Cleat has a mesh base for excellent retention. The Buttons also have a mesh base  and are available in different shapes and with eyelet.

They can be bonded to the vestibular or lingual side of the tooth using orthodontic adhesive. MIM-made Hooks and Stops are designed to create an anchorage or stop point on the arch. Open hooks are universal, while closed ones are available for 0.018" and 0.022" slots.

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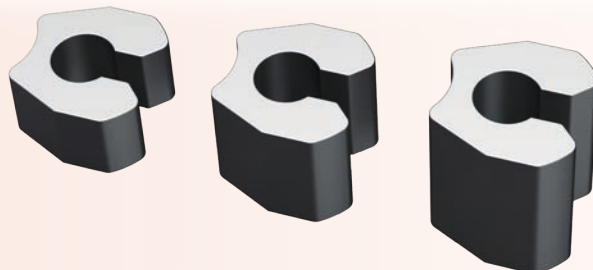
ACTIVA is a product that allows the reactivation of the open springs in a simple, practical, and extremely rapid way. With **ACTIVA**, once the space required for spring reactivation has been quantified, it is only a matter of compressing it towards one of the two sides and insert the **ACTIVA** of the desired thickness. In this way, the spring, compressed again, reactivates its thrust function.

ADVANTAGES

– Cooperation with the sliding mechanics: unlike the common crimping devices, Activa is not crimped onto the wire but around it, allowing the wire to move freely without compromising the sliding.

– Flexibility: selection of the needed compression level thanks to three available sizes. Activa fits on any

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size archwire and can be used anytime during treatment. It is also possible to increase the activation of the spring progressively by adding additional sizes of ACTIVA.

– Saving materials: no need to change the existing spring.

–Reduced chair time: spring replacements are no longer needed, which results in less chair time.

USE OF ACTIVA

Step 1:

Choose the correct size of ACTIVA is needed;

Step 2:

Grasp ACTIVA using any plier.

With an elastic ligature director compress the spring to one side;

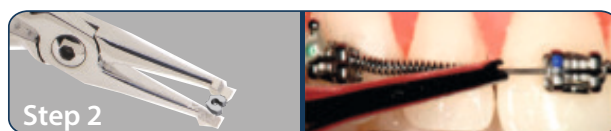
Step 3:

Insert ACTIVA on the archwire and crimp it gently. Release the spring.

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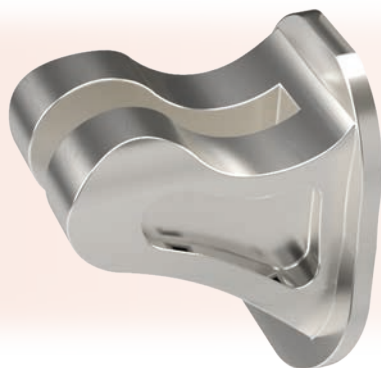
BT2 is the key for rapid correction of severe skeletal and deep overbites. Its patented curvature reduces muscle hyperactivity and parafunction, permits anterior guidance of the lower incisors and prevents anterior interferences during forward grinding.

BT2 is wear-resistant, has a new and unique vertical groove to help placement, and facilitates debonding as it is simply collapsable.

BT2 allows a physiological eruption of molars and premolars without the effects of molar intrusion. Its wide and anatomical mesh base ensures a high resistance to debonding, frequent in this type of device due to interference in the occlusion.

BT2 is made of stainless steel that avoids the

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ingestion of toxic substances, allows having a product of immediate use, reduces the chairside time, and leads to solve deep overbites in 4-6 months in both children and adults.

USE OF BT2:

1. *Green stone pellicle layer removal over enamel secures palatal bonding, followed by pumice, etc etc. (Fig. 1). Patient is asked to position their head up and back, for palate to be as perpendicular to floor as possible.*
2. *Position the base of the BT2 at about 75° angle to the J-shape of the bracket holder (Fig. 2) with larger base curvature to the incisal edge), and tapered end of bonding pad toward the gingival (Figs. 3, 4).*
3. *Ideally use the Periodontal Probe or the thicker neck of Hollenback instrument in the BT2 split opening to seat BT2 (Fig. 5) finger pressure may also be used to initially adapt BT2 to palatal curvature of incisors (Fig. 6).*
4. *Use articulating paper for BT2 occlusal adjustment with green stone for balance, if required (Figs. 7,8).*

Fig.1

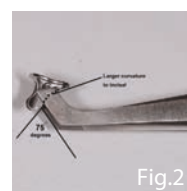
Fig.2

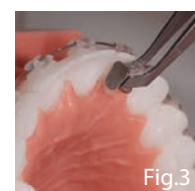
Fig.3

Fig.4

Fig.5

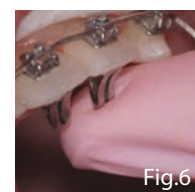
Fig.6

Fig.7

Fig.8

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TS2 is a new improved device for the Rapid (4-6 months) Open Bite Correction.

Main features

- *Nine rounded reminder protrusions prevents tongue thrusting (TS2 do not cut tongue, operator's glove or skin);*
- *80-gauge mesh bonding pad for increased shear bond strength;*
- *Six Undercuts for cross bite elastics in lateral open bite treatment that commonly require maxillary expansion;*
- *Very small tongue inhibitor for good oral hygiene.*

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**Why to use TS2**

- *Rapid Anterior and Lateral Open Bite Closure.*
- *Prevents anterior-lateral tongue positioning.*
- *In lateral open bites (Fig. 2A, 2B), undercuts are useful additionally for 1/4", 6oz heavy crossbite elastics at premolars, molars.*

When to place

- *In early and adult treatments with orthodontic brackets.*
- *At the second appointment after brackets have been placed.*

How TS2 works

- *Tongue retracts from forward positioning using a negative conditioning response, similar to a hot-stove reflex of the tongue (Fig. 3). TS2s are reminders and do not cut tongue, or operator's gloves or skin.*

Where to place

- *Palatal of upper anteriors 3 to 3 and Lingual of lower anteriors 3 to 3, bonded just below the center of the crown to permit open bite closure (Fig.4).*
- *Bonded to the premolars and molars for lateral open bites.*