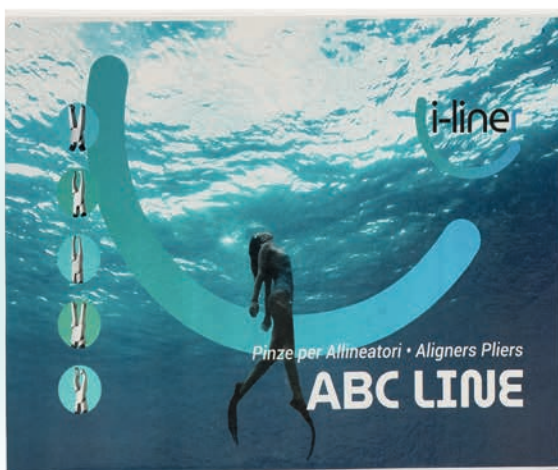
STRUMENTI *INSTRUMENTS*

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The "medical grade" stainless steel component of Sia's instruments ensures an high resistance to corrosion.

The most advanced technology allows a smooth edges and a precise tip aligned product. Sia's pliers are one year guarantee on production defects, joint malfunction, legs breakage and inserted tip separation. For sterilization all well known methods are accepted, taking care on the corresponding directions included in the package.

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Plier to make round cuts to create room for other devices (tractions such as lingual buttons, eyelet, TS2). To create a space on a specific tooth, place the plier on the lingual side of the aligner, then press to cut.

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Plier to make a drop cut, useful in treatments where the use of the elastics is required. Place the plier on the aligner's edge and press sufficiently to cut it. The cut created is an anchor for applying the elastics directly on the aligner.

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Plier to make a rectangular pressure point on the aligner's mesial or distal side. Place the instrument, mesially or distally, where you want to make the pressure point and press to shape the aligner. Use the lateral key to regulate the depth of the pressure point.

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Plier to make cylindrical pressure points on the aligner's mesial or distal side. Place the instrument, mesially or distally, where you want to make the pressure point and press to shape the aligner. Use the lateral key to regulate the depth of the pressure point.

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This plier makes bite plans on the aligner. Use the lateral key to regulate the depth of the bite plan. Heat the plier on a flame before using it. Once heated, place it on the lingual side of the aligner, and press to form the bite plan.

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Specially designed to cut hard wires up to 0.022"x0.028", with tungsten carbide inserts.

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A non angled pin cutter designed to cut soft wires up to 0.016". Made of high speed stainless steel.

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Tungsten carbide inserts for a more effective cutting operation together with the holding of the wire chopped pieces. The high-quality steel guarantees a long life of the instrument. Suitable for wires up to 0.022"x 0.028".

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A general utility plier with 3/32" diameter when tips are serrated. Designed with rounded tips for maximum patient comfort and safety

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A general utility plier similar to the previous one with 3/32" and an offset angle of 45°. Matching serrated tips hold wires firmly. Maximum wire diameter 0.015".

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Specially serrated tip design ensures the insertion even between the smaller brackets' tie wings, aiding an easy and safe seating of the bands.

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Designed to crimp auxiliary stops and hooks to archwires. The slotted tip fits over the base of the hook while holding it tight during crimping.

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Designed to crimp auxiliary stops and hooks to archwires. The special design of the tips allows an easy positioning of the auxiliaries even in the mouth's posterior regions.

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A general utility plier featured of tapered tips and of a convenient working angle. Very suitable to access to posterior regions.

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Makes 1mm V-Bends on archwires avoiding wire scoring and so no breakage. Works on round arch wires up to 0.020".

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Wire bending plier. Orthodontists can use this plier in many ways: flat pliers, wire cutters, fixing pliers.

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The gradual conic design of the tips allows to bend wires up to 0.030".

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Designed to bend round wire up to 0.030". It is provided of one conic and one pyramidal beak.

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Designed to bend round wire up to 0.016". The long tapered beaks allow an easy bending even of small diameter loops.

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Ideal for quickly forming metal ligatures.

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The use of this plier reduces working time. Special double sided offset head forms right hand or left hand step-up or step-down bends without removing the archwire from the slot.

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Especially designed with concave-convex beaks to make bull and tie loops on rectangular wire up to 0.020".

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Special designed to forms and bend with precision round and rectangular wires in a very short time. The smooth and round surface of the beaks allows an easy shaping.

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Beveled edges prevent scoring of wire. Four steps allow precision forming of 3,4 and 6mm loops even on rectangular wires.

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Especially designed to form and bend loops on rectangular wires. Provided with a cylindric tip of progressive diameters ("0.045"; 0.060"; 0.075") and of grooved a concave beak. It bends wire up to 0.022"x0.028".

**PINZA TWEED PER PIEGARE ARCHI RETTANGOLARI ~
RECTANGULAR ARCH FORMING PLIER, TWEED STYLE**

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Used to make offset and bend of 90° on rectangular and square wires up to 0.022"x0.028". The special beaks do not score the wire while working.

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Ideal for precise bending of lighter wires. Precision grooves ensure accurate bending and closing of loops.

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Designed to form lingual archwires. Perfect solution even on wire up to 0.036".

PINZA "DE LA ROSA" ~ "DE LA ROSA" PLIER

Per contornare o formare archi.

To form archwires in every working position.

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With grooves to form archwires in every working position.

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Special laboratory plier used for wires up to 0.045".

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Young plier to form loops of different diameters.

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Designed for an easy removing of the brackets. The wide tips wedge between both edges of the base of the brackets and the tooth.

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For easier and more comfortable removal of the bands. The removable pad rests on the occlusal side of the molar, and the tip fits under the band. By removing the pad, the instrument is autoclavable.

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Provided with a narrow beak it is a solid and simple pliers, perfect to tie and to bend wires. Ideal for elastic ligature and chains placement. Provided with a lock-on handle.

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Provided with a hooked tip, it is a solid and simple utility plier, perfect to tie elastic and metal ligatures. Provided with a lock-on handle.

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The beak's angulations allows an easy placement of the separators. It is special designed to expand and reduce modules for an easier an quick application.

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Useful to check the distance between the bracket's slot and the incisal tooth edge.

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Sterilizzabile in autoclave a 120°C – 30 min.

REF: D30-IPR

The IPR Gauge is an autoclavable stainless steel instrument used during Interproximal Reduction (IPR) to measure space. The IPR Set includes 6 gauges: 0.1 mm, 0.2 mm, 0.25 mm, 0.3 mm, 0.4 mm, and 0.5 mm.

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Up to 250°F/125°C high temperature resistance. Special designed for an easy access to the second molar and with grooves to better eliminate the occlusal interference problem and so reducing the risk of abrasion for patients.

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Aluminum band pusher with stainless steel tip; a precise and versatile tool for the band positioning and modeling procedure. The tip design, studied to modify the bands in the intercuspid grooves, allows shaping the less precise edges, making the band adherent to the tooth's anatomy. The rubber cap protects the patient from abrasion and is removable and autoclavable.

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Handy and lightweight, it allows easier positioning of the bands.